

Aldehyde Free

# Mi-Silperox<sup>®</sup>

Eco friendly, Surface disinfectant & Air disinfectant



- Eco friendly-Decomposes into water and oxygen.
- Non carcinogenic and non mutagenic.
- Biodegradable.
- Does not change pH on dilution.
- Non eroding.
- No rinsing required.
- Less OT shut down time hence saves time and financial loss.
- Broad spectrum-Kill all pathogens including spores & micobacterium.
- Resistant Strains like MRSA VRE.

Active ingredient  
0.1% w/v Silver Nitrate IP  
10% w/v Hydrogen Peroxide IP



**FMS**  
You Say. We Disinfect.



## Eco friendly, Surface disinfectant & Air disinfectant

### Direction :

**Surface Disinfection :** Use 1 :10 **Mi-SILPEROX** i.e. 10 ml **Mi-SILPEROX** + 90ML DM water.

**Air Disinfection :** Use 1:5 **Mi-SILPEROX** i.e. 20ml **Mi-SILPEROX** +80 ML DM Water.

**Sprayers :** 1000 cu ft area room is ready in 60 minutes with only 20% **Mi-SILPEROX**.

**Mi-SILPEROX** is a clear solution & complex formulation of processed silver, a Noble metal and highly concentrated Hydrogen Peroxide.

The Hydrogen Peroxide component ensures a rapid germ killing effect and the silver component ensures that effect is long lasting.

Silver is well known for its cold sterilization.

The effect of two components is multiplied by a special manufacturing process of **Mi-SILPEROX** giving a synergistic effect.

**Mi-SILPEROX** is a clear universal disinfectant based on a complex of a silver ions and hydrogen peroxide as an oxidizing agent that can be used for air, surface and water disinfection.

**Mi-SILPEROX** does not alter colour, taste or smell of food/water and leaves no residue.

**Faith micro solutions recommends use of Mi-Fogger ULV Fogger for environmental decontamination.**

### Precautions :

- For external use only Avoid direct contact with eye.
- Flush eyes with fresh water in case of contact.
- In case of emergency, seek medical help.
- Contact with aldehyde based products must be avoid if disinfection was preciously carried out using aldehyde based product, intermediate cleaning is recommended before changing to Mi-Silperox.
- Inhalation : Move to fresh air immediate. If experiencing difficulty breathing seek medical attention.
- Eye contact : In the case of contact with eyes, wash immediately with plenty of water and seek medical attention.
- Skin contact : In the case of contact with skin wash immediately with plenty of water and seek medical attention.
- Keep out of reach of children.

**Pack Size : 5 Ltr. & 1 Ltr.**



You Say. We Disinfect.

### FAITH MICRO SOLUTIONS

ISO 9001 : 2015 & ISO 13485 : 2016 Company

Head office: 27, Ashoka Chambers, 5-B Rajendra Park,  
New Delhi-110060, INDIA

Phone : 011-47479797 (20 Lines)

Customer Care : info@faithmicrosolutions.com

Url : www.faithmicrosolutions.com

Authorised Representative :

Connect with us at :

